

RS PRO Burn Plasters – Pack of 10 in Two Assorted Sizes

RS Stock No.: 291-921

Features

- Individually wrapped sterile gel plasters
- Slight cooling effect soothes minor burn injuries
- Transparent polyurethane gel dressing allows healing process to be observed
- Hypoallergenic adhesive polyurethane film border prevents bacterial penetration and leaves no residue upon removal
- Pack of 10 plasters - 5x 75mm x 100mm, 5x 45mm x 65mm



RS PRO is the own brand of RS. The RS PRO Seal of Approval is your assurance of professional quality, a guarantee that every part is rigorously tested, inspected, and audited against demanding standards. Making RS PRO the Smart Choice for our customers.

Product Description

The HypaSoothe Burn Plasters deliver cooling action to injuries, ideal for use on minor burns and scalds. They are supplied in a pack of 10 plasters of assorted size (5x 75mm x 100mm and 5x 45mm x 65mm).

They are effective at creating a moist wound healing environment and have a slight cooling effect to soothe and relieve pain. The transparent polyurethane film border protects the wound from bacterial contamination.

How to Use:
Immediately cool the burn under running water for at least 20 minutes. Pat the area dry gentle before applying a plaster. Ensure you choose a size plaster that is big enough for the wound/injury. The polyurethane film should only come into contact with the skin and not the burn wound/injury. Replace with a new plaster at least once a week or more often if required. The healing process can be observed without removing the plaster. To remove the plaster, press gently down on the skin and carefully lift the polyurethane film around the burn, then carefully lift the plaster away.

Hazards and Cautions:
To help prevent infection, wash hands before use and after dressing the burn injury.
Our HypaSoothe Burn Plasters should not be used on third degree burns or highly exuding wounds. Do not use on large or severe burns or on burns that are infected or bleeding. Seek medical advice. If the burn becomes reddened, swollen or does not appear to heal after a period of time, seek medical advice.
After a period, the plasters will dry out; this is normal.
Remove and replace the dressing if necessary.
Do not use if you have any known reactions to polyurethane gel.
Do not used if the wrapper is damaged or it's dried out.
Do not use after expiry date.
Do not re-use plasters, these are single -use only and will negatively affect performance and greatly increase risk of infection.
Plasters are sterile until opened.

General Specifications

Product Dimensions	12.3 x 1.5 x 20.5 cm
Product Weight (grams)	70
Wall Bracket Included	No
Medical Device Class	Class I Sterile
Application	Burn Care
Suitable for	Providing initial protection for small burn injuries
Shelf Life	4 years from manufacture



Classification

Commodity Code	3005100000
----------------	------------

Classification