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Globus (Shetland) Ltd, T2 Trafford Point, Twining Road, Shetland, ZE2 5JL, UK

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Dissolution levels indicate the change in Drug:stearate Resistance of the glass after exposure to the challenge chemical.

Degradation levels indicate the change in Puncture Resistance of the glove after exposure to the challenge chemical.

EN ISO 374-5:2016

Performance Level	AQL	Inspection Levels	Penetration leak: PASS Penetration water leak: PASS Performance level: Level 2
Level 3	< 0.65	G1	
Level 2	< 1.5	G1	
Level 1	< 4.0	S4	Micro-organisms risk in accordance with EN ISO 374-5:2016

**VIRUS** Protection against bacteria and fungi: PASS  
Protection against viruses: TESTED  
  
Products conforms to the requirements of EU regulation 2016/425 of the European Parliament of the Council as Personal Protective Equipment (PPE). Type examination certificate Module B issued and Module D assessment carried out by: EU: SATRA TechnoLogy Europe Ltd, Brackenbus Business Park, Cheadle, Dublin, D15 YN2P, Ireland (Notified body No. 2777).

## Intended: Medical and PPE Cat III

Comply to MDR 2017/745 Class I  
2777 Comply to PPE Regulation 2016/4

**STATEMENT AND CAUTION** Test results are taken from palm area of the glove. Glove performance quoted is based on laboratory data and may not reflect the actual duration of protection in the workplace due to the differentiation between mixtures and pure chemicals and other factors in fluency the performances such as temperature, abrasion, degradation etc. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only and relates only to the chemical tested. It can be different if the chemical is used in a mixture. When used, protective gloves may provide less resistance to the dangerous chemicals due to changes in physical properties. Movement, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. There are no possible allergens within the glove that known to cause harm to the wearer. Before usage, inspect the gloves for any defects or imperfections.

A horizontal row of three icons enclosed in a thin black border. The first icon is a white square containing a black outline of a factory building. The second icon is a white square containing a black outline of a pipe with a wavy, winding path. The third icon is a white square containing a black outline of a valve or flange.

**GLOBUS**

**Globus (Shetland) Ltd.**, T2 Trafford Point, Twining Road, Trafford Park, Manchester, M17 1SH, UK | T: +44 (0)161877 4747 | [www.globusgroup.com](http://www.globusgroup.com)  
**Globus EMEA FZE**, Jafza View 1, Office 2201, Jebel Ali, PO Box 61195, Dubai, United Arab Emirates | T: +971 4 8829962

**EU REP** Globus EMEA Ltd Authorized Representative in the European Community

Globus EMEA Ltd., 51 Dawson St, Dublin, D02 AN25, Ireland

A small rectangular logo containing the text "MADE IN MALAYSIA".

MADE IN MALAYSIA